***GE Medical Systems***

---

**Direction 2174206**

**LOGIQ 500/400  
Advanced OB Calculations  
Software Option**

**H4550KY**

This page intentionally left blank.

**GE Medical Systems**

---

# **Technical Publications**

**Direction 2174206**

**Revision 3**

## **LOGIQ 500/400 Advanced OB Calculation Software Option Installation Manual**

Copyright © 1997, 1998, 1999 By General Electric Co.

**Operating Documentation**

---

**GE Medical Systems**

*GE Medical Systems: Telex 3797371  
P.O. Box 414, Milwaukee, Wisconsin 53201 U.S.A.  
(Asia, Pacific, Latin America, North America)*

*GE Medical Systems — Europe: Tel: +49 (0) 212 28 02 208  
Beethovenstraße 239, Postfach 11 05 60, D-42655 Solingen  
GERMANY*

## Safety Warnings

### WARNING

- This Service Manual is available in English only. If a customer's service provider requires a language other than English, it is the customer's responsibility to provide translation services. Do not attempt to service the equipment unless this service manual has been consulted and is understood. Failure to heed this Warning may result in injury to the service provider, operator or patient from electric shock, mechanical or other hazards.

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## Revision History

REV	DATE	REASON FOR CHANGE
0	May 1, 1997	Initial Release
1	June 17, 1997	Correct Functional Checkout Procedure
2	September 29, 1998	Correct Error in Option Name Page 2
3	October 21, 1999	Add Option Numbers to Change

### LIST OF EFFECTIVE PAGES

PAGE NUMBER	REVISION NUMBER	PAGE NUMBER	REVISION NUMBER
Cover Page			
Title Page	3		
Language Warnings i and ii	3		
Revision History A and B	3		
Procedure 1 thru 2	3		

Please verify you are using the latest revision of this document. Information pertaining to this document is maintained on GPC (Global Product Configuration). If you cannot access GPC, contact your Service Support Center.

This page left blank intentionally.

# SECTION 1 – Installation

## 1-1 Introduction

LOGIQ™ 500/400 Software Options which include PDI (Power Doppler Imaging) Package, OB Calculations Package, Vascular Calculations Package, Advanced OB Calculations and Advanced Vascular Calculations are shipped from GE YMS turned off.

Shipping systems with these software packages turned off allows for tighter inventory control since we can deal with importing only one standard configuration.

Options NOT purchased by the customer should NOT be turned on.

## 1-2 Tools Required

NONE

## 1-3 Time Required

Approximately 30 minutes.

## 1-4 Parts/Documentation Required

1.	LOGIQ™ 500 User Documentation	
2.	LOGIQ™ 500 Service Manual	P9030TA
3.	LOGIQ™ 500 Proprietary Manual	P9030TB
4.	LOGIQ™ 400 User Documentation	
5.	LOGIQ™ 400 Service Manual	2127661
6.	LOGIQ™ 400 Proprietary Manual	2127662

## 1-5 Functional Check-out

Perform functional checks to verify that the system is functioning properly prior to installing any peripherals. Refer to the Functional Checks in Chapter 4 of the Service Manual.

## 1-6 Procedure

READ and UNDERSTAND these instructions thoroughly before proceeding with the installation. Perform each step in sequence and check it off when completed. If a problem occurs after completion, go back through the procedure and check for implementation errors before contacting your Ultrasound Region Engineer or the Online Center for assistance.

1. Ensure that all other installation procedures are completed and determine that the LOGIQ™ 500/400 is functioning properly prior to enabling (turning on) any authorized software options.

### Turn on Advanced OB Calculations

2. Refer to the LOGIQ™ 500 Proprietary Manual (P9030TB) or LOGIQ™ 400 Proprietary Manual (2127662), section 2–4 for information on how to enter the service menu and select **03 Option Set**.
3. Refer to the LOGIQ™ 500 Proprietary Manual (P9030TB) or LOGIQ™ 400 Proprietary Manual (2127662), section 2–4–2 for information on how to install the software options purchased by the customer. For Advanced OB Calculations ensure that option number **22** and **25** are turned on.

### Final Checkout

1. Check to ensure that all option packages installed are functional.
2. Refer to the LOGIQ™ 500 and LOGIQ™ 400 and ensure that the Advanced OB Calculations package functions normally.
3. When all functionality is verified, the software option installation is complete.